Bovia Medical  
5115 Ulmerton Road  
Clearwater, FL 33760 USA

# Reusable Grounding Cable

REF: A1202C

Qty. 1

CE

SEE INSTRUCTIONS FOR USE

Rx ONLY

## Instructions for Use Reusable Grounding Pad Cable

**WARNING:** Inspect prior to use. Do not use if cable insulation is cracked, split, or otherwise damaged. Electric shock could result.

**CAUTION:** Do not autoclave. Damage may occur.

**CAUTION:** Do not use a reusable (steel) grounding plate with A1202C or A1252C cables. Breakage of the clip may occur.

**Use:** Grounding pad cables are used to create a pathway to direct the electrical current through the patient back to the generator. Cables are only compatible with specific generators and grounding pads.

**To Clean:** Wipe with a cloth or gauze that has been soaked in warm water and dry with a clean cloth. Disinfect using a phenol or glutaraldehyde based solution prepared according to the manufacturer's recommendations. Wipe again with warm water and dry using a clean cloth.

**CAUTION:** Do not soak the cable in any disinfection solution as damage may occur.

Cable Catalog Number	Grounding Pad	Generator	Use
A1202C	A1202 (Single-Use)	A950	Open Blue clip and heat grounding pad. Close clip by pressing downward until the lid clicks into place. Do not force the clip to close.
A1252C	ESSE, ESSE (Single-Use)	A1250, A2250, A3250, IDS-200, IDS-300	Align male end of cable with female end of grounding plate. While holding the cable, turn the plate clockwise until the plate is securely attached.
A1254C	All reusable (steel) grounding plates	A1250, A2250, A3250, IDS-200, IDS-300	

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