



Delasco

608 13th Avenue
Council Bluffs, IA 51501

Dermatologic Lab & Supply, Inc.

Email: questions@delasco.com

Website: www.delasco.com

1-800-831-6273 including P.R. & Canada

Local & International: 1-712-323-3269

24-Hour Fax: 1-712-323-1156

DELASCO ELECTRICATOR® FOOTSWITCH: PART #26500

CAUTION: FEDERAL LAW (USA) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN OR HEALTH CARE PROFESSIONAL.

CAUTION: THIS DEVICE IS SUPPLIED NON-STERILE.

CAUTION: DO NOT USE THIS DEVICE IN THE PRESENCE OF FLAMMABLE MATERIALS.


WARNING: READ ALL ELECTRICATOR DEVICE INSTRUCTIONS PRIOR TO USING THIS DEVICE.

WARNING: MAKE SURE POWER TO THE DELASCO ELECTRICATOR UNIT IS OFF WHEN INSERTING FOOTSWITCH PLUG INTO UNIT. FAILING TO TURN UNIT OFF BEFORE INSERTING THE PLUG MAY RESULT IN DAMAGE TO THE UNIT.

Inspection

Inspect the footswitch upon receipt. If damage has occurred during shipment, do not use and notify Delasco immediately.

Instructions for Use and Reuse

1. The Delasco Electricator® Footswitch is the only footswitch designed to be used with the Delasco Electricator®. Do not attempt to use this footswitch on any other unit, or any footswitch with the Delasco Electricator®.
2. With power off, insert the male end of the footswitch completely into the receptacle marked with the  symbol at the bottom of the unit. The connection must be secure for proper function.
3. Make sure either Delasco 2300 handpiece or Delasco 26409 bipolar cord with bipolar forceps is connected to the Delasco Electricator®.
4. To activate the footswitch, switch power on, select mode and power setting. Footswitch will now activate either the handpiece or bipolar forceps.
NOTE: The 26300 handpiece may be activated by using the footswitch or the activation button on the handpiece. The handpiece should be stowed in the Delasco Electricator® handpiece cradle when not in use.
5. Footswitch is not intended to be sterilized. Doing so may damage the footswitch or cause an unsafe condition.