



**Delasco**  
608 13<sup>th</sup> Avenue  
Council Bluffs, IA 51501

**Dermatologic Lab & Supply, Inc.**

Email: [questions@delasco.com](mailto:questions@delasco.com)

Website: [www.delasco.com](http://www.delasco.com)

1-800-831-6273 including P.R. & Canada

Local & International: 1-712-323-3269

24-Hour Fax: 1-712-323-1156

## **DELASCO UNIVERSAL ELECTROSURGERY WALL MOUNT KIT**

### **PART # 26600**

#### **Inspection**

Inspect the wall mounting kit upon receipt. If damage has occurred during shipment, do not use and notify Delasco immediately. Kit should contain: mounting plate (A), qty =1, plastic anchors (B), qty = 2, and Phillips mounting screws (C), qty = 2. Read all instructions completely.

Delasco is not responsible for improper/incorrect placement that may result in damage to any medical device. Check screws on back of electrosurgical unit to be mounted. Make sure screws are large enough to hold unit firmly on wall bracket.

#### **Tools Needed**

Phillips screwdriver, hammer

#### **\*\*IMPORTANT\*\***

**SELECT A MOUNTING LOCATION THAT IS:**

1. Convenient for use.
2. Near an electrical outlet.
3. At a point on the wall where a secure mount may be achieved.

#### **Mounting Instructions**

1. Hold mounting plate (A) (open channels up) against the wall on selected site. Make sure plate is level and mark mounting hole locations onto the wall.
2. If unable to utilize wall stud, take the plastic anchors (B) and lightly tap them into place about ½ inch into the drywall. If utilizing wall stud, one plastic anchor (B) will not be used.
3. With Phillips screwdriver, firmly drive plastic anchor(s) (B) into the drywall until flush with wall (note: anchors are not designed to be driven into wall studs).
4. Put mounting plate (A) in place, insert mounting screws (C), and tighten until secure.
5. Carefully slide electrosurgical unit in place on the wall bracket. Screws on back of electrosurgical unit may need adjusted to firmly hold unit against mounting plate.

