



Delasco
608 13th Avenue
Council Bluffs, IA 51501

1-800-831-6273 including P.R. & Canada

Local & International: 1-712-323-3269

Dermatologic Lab & Supply, Inc.

Email: questions@delasco.com

Website: www.delasco.com


24-Hour Fax: 1-712-323-1156

DELASCO UNIVERSAL MOBILE STAND: PART NO. 26800

CAUTION: FEDERAL LAW (USA) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN OR HEALTH CARE PROFESSIONAL.

Inspection

Inspect the Universal Mobile Stand upon receipt. If damage has occurred during shipment, do not use. Notify Delasco immediately.

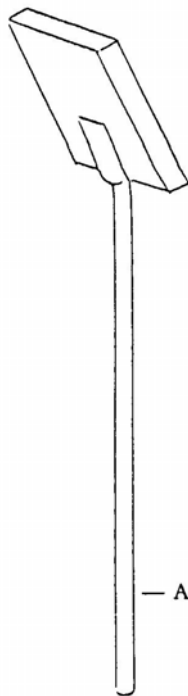

POST

Assembly Instructions

- 1.) Position post (A) into hub of base.
- 2.) Insert flat washer (B), lock washer (C), and screw (D) through the bottom of base. Turn until snug and tight.
- 3.) Note: The Universal Mobile Stand requires the 26600 Wall Mount Kit (sold separately) in order to attach the electro-surgical unit to the mobile stand.

