

LightSpeed user instructions

Directions for use:

! **Warning:** Read all instructions and warnings before using this product; Failure to follow these instructions and warnings may result in serious personal injury, or serious illness, including blindness.

Before using the LightSpeed eyewear:

1. To ensure a long battery life, the battery has not been activated. To properly activate the battery, remove the strap extending from the battery compartment.
2. Hold the LightSpeed eyewear in your hand, with the front of the eyewear facing the IPL-flash, but with your back to the IPL-flash (to avoid you looking into the flash; make sure you do not obstruct the path of light to the LightSpeed). Ask a colleague to operate one pulse of light, and check that the eyewear darkens immediately.

Note: A special solar-powered clicker is available with which you can test the LightSpeed. Ask the company from whom you purchased the LightSpeed for further details.

! WARNING

Do NOT use the eyewear if it does not immediately darken in response to the IPL-light. In such a case, use of the eyewear may cause serious personal injury, including blindness.

3. Adjust the temple pieces and the safety strap to ensure a comfortable, secure fit.
4. You are now ready to use the LightSpeed eyewear with your IPL device. ALWAYS face (or look towards) the IPL device/hand piece so that the sensors detect the light. LightSpeed will react within 0.3 milliseconds to the light. Once the IPL light is off, it will take 0.5 seconds for the eyewear to return to the 'light' position. Should it take longer for the LightSpeed to react to the light and darken, change the battery immediately.

If the LightSpeed is not responding to the light:

1. Check to ensure the battery is securely placed in the battery compartment.
2. Battery should last approximately one year. We recommend that you change the battery every 12 months.
3. After checking and changing the battery, test the eyewear again. IF the product still does not respond, contact the company from whom you purchased the device.

The warranty on this product does not cover any product or component found by Bacou-Dalloz to have been altered, misused, abused, tampered with or improperly maintained. The warranty on this product does not cover any damage, loss or expense due to damage caused by accident, abuse, improper maintenance, use or misuse of the product.

How to change the battery:

1. Remove the temples by holding your thumb on the bottom of the temple close to where the temples attach to the front lens. Using your forefinger, press down, then slide the temple away from the front lens. Repeat to remove the other temple.
2. Remove the two inner side panels.
3. Remove the front inner panel; Turn it over so that the battery is facing you. Remove battery. Place new battery into battery compartment ensuring the battery is securely in place.

WIP# 353900G-01 (9/05)

4. Place front panel back into front clear lens. Battery should face towards the inner blue front. Next replace the two side panels. Note there is a right and left panel, they are indicated with R or L.

5. Starting on the right side, place shaft of temple into bottom hole of the front lens. Place thumb on bottom of temple, use your forefinger to depress top of temple and insert into the top hole on the front lens. Repeat for the left temple.

6. Check to ensure your LightSpeed responds to the IPL-light, by following steps 2-3 under 'Before using your LightSpeed eyewear' above.

Cleaning and Care instructions

Clean with a soft cloth only.

! CAUTION

Cleaning with water or alcohol can damage the product, cause the eyewear to malfunction and void the warranty.

To extend product life, do NOT place on a surface with the lens face-down, as scratches to the lens may occur. The product should be kept in the pouch it was supplied in at all times when not worn.

! WARNING

- Read all instructions and warnings before using this product. Failure to follow these instructions and warnings may lead to serious personal injury, or serious illness, including blindness.
- Do not make any modifications to this product.
- Do not use this product for protection against laser light or welding.
- This eyewear should be inspected before each use. If it has worn, damaged or broken parts, then it must be replaced immediately.

Manufacturer : Glendale Inc., 10 Thurber Blvd, Smithfield RI 02917, USA
Certification office : Dincertco GmbH, Germany, notified body nr. 0196
European sales office : GPT Glendale Inc, Christofsstr. 5, D-55116 Mainz

CE certification according to the European Council Directive 89/686/EEC.