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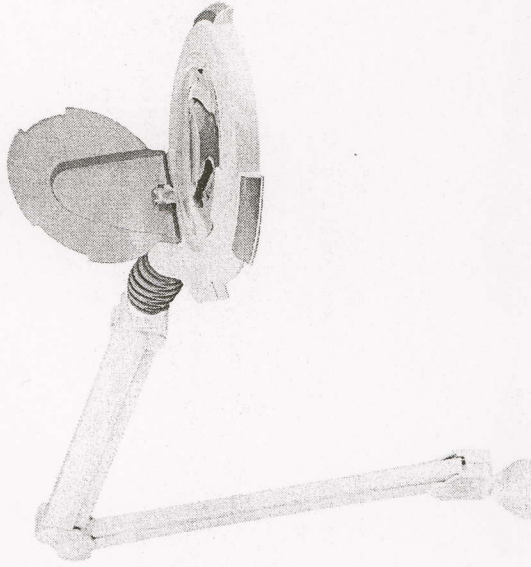
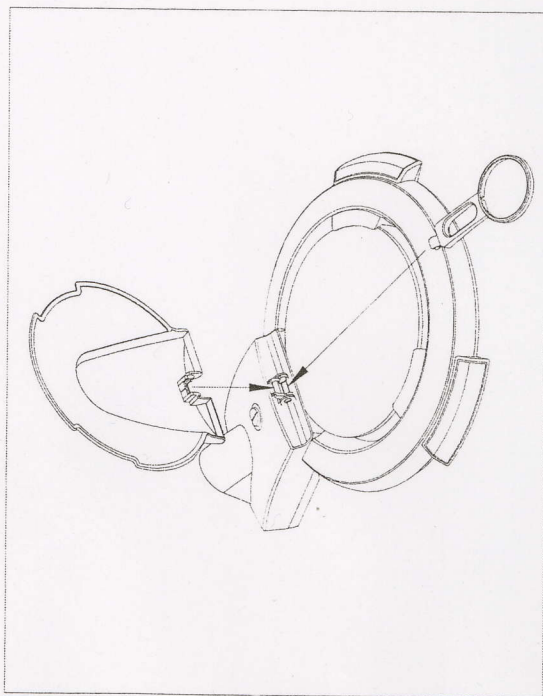
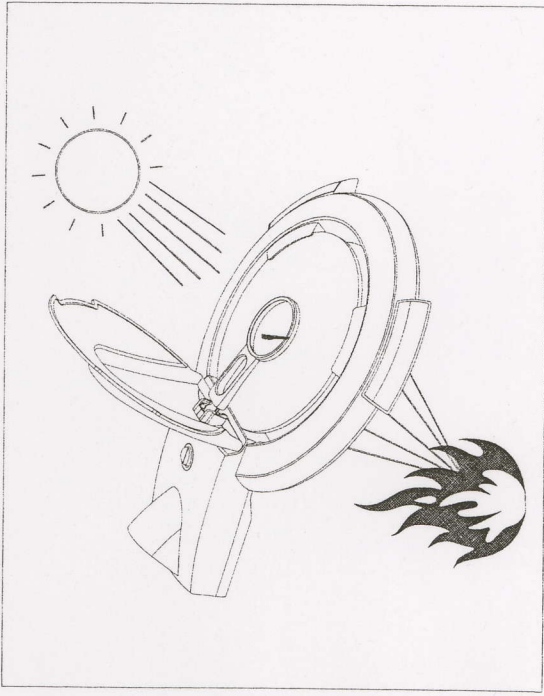
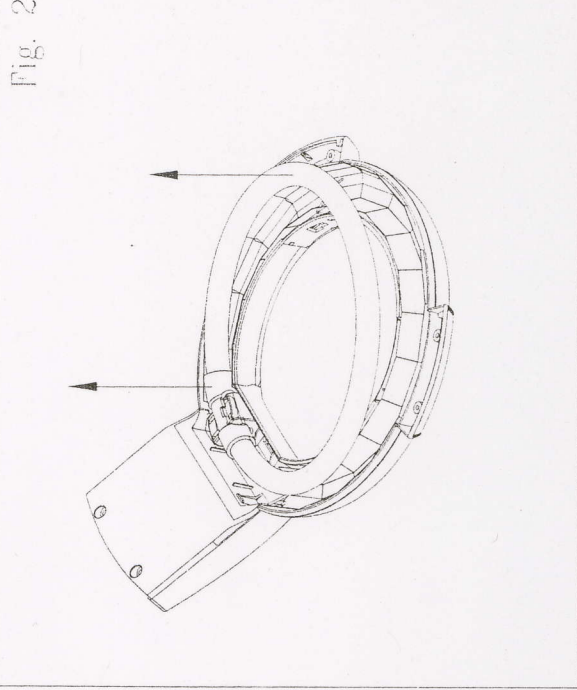
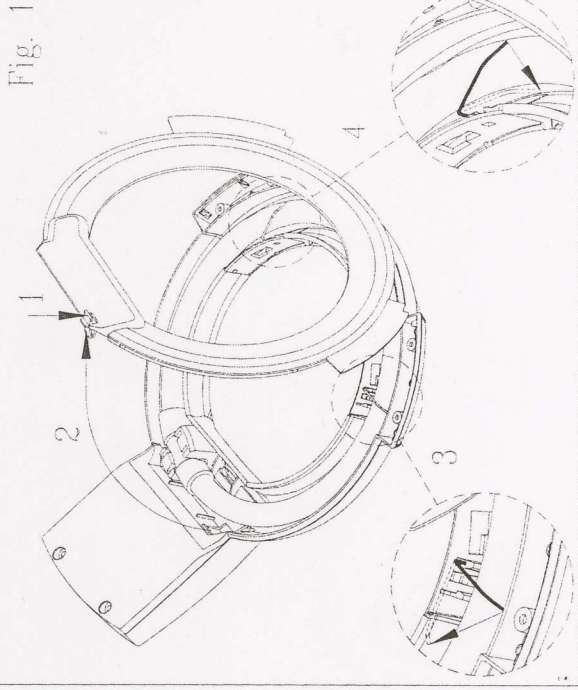
ART.: 05-900-695-00 Page 3 of 8  
BRU-303-0013 Rev.nr.: C

## ENGLISH

Instructions for use – Circus and Circus Medical  
Illuminated magnifier – 22 W

### General information

Magnifying lamp with circular T5 light tube for quality-conscious users. Used by beauty salons, doctors, skin care specialists, etc.

### Electrical connection

The power circuit includes an electronic ballast that supplies a circular T5 light tube. The power cable must only be replaced by a qualified electrical fitter or other similarly qualified professional.

### Replacing the light tube

Start by switching off the power at the mains and removing the plug.

1. Remove the light tube cover (see fig 1, pos. 1 and 2)
  2. Press the springs that hold the light tube in towards the lens (see fig. 1, pos. 3 and 4)
  3. Use both hands to remove the light tube (see fig. 2)
- Fit the new light tube by reversing the above procedure.

### Cleaning

Open the lamp head as described above. Blow and dry off any dust and dirt.

Using of different cleaning agents for cleaning this product:

Avoid use of alcohol, acids or any kind of organic solvents (Isopropanol etc.)  
Only use the weak mix of ordinary soap and water without any perfumes.

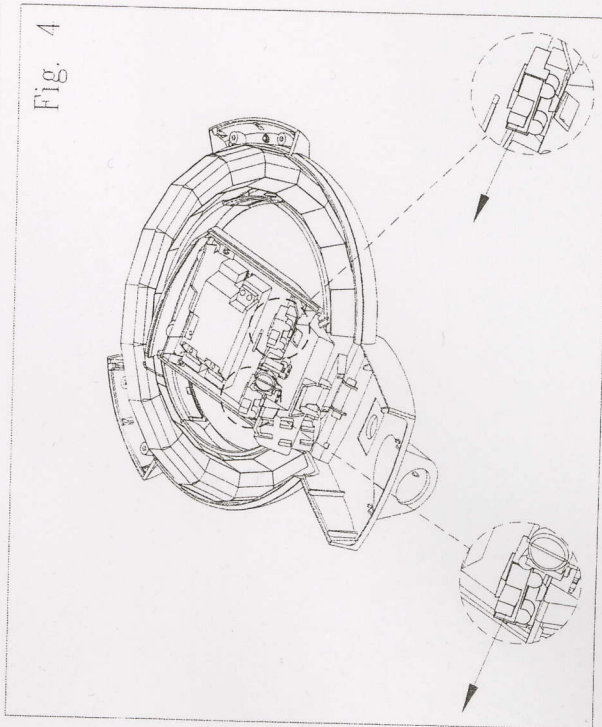
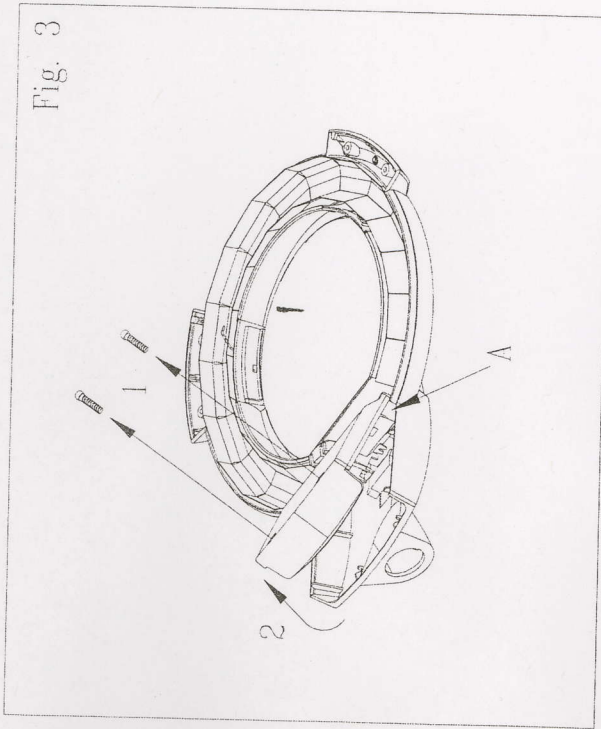
### Troubleshooting

1. The light tube does not light up
    - Replace the circular T5 light tube
  2. For qualified electrical fitters and other similarly qualified personnel
    - Check the mains connection
    - Replace the fuses (NB! Only applies to CIRCUS MEDICAL)
      - ⇒ Remove the light source (see the section concerning replacement of the light source)
      - ⇒ Remove the screws on the underside of the lamp head (see fig. 3, pos 1)
      - ⇒ Lift up the cover where the cable joins the lower section of the arm (see fig.3, pos. 2)
      - ⇒ Remove the fuse holder, change the fuse and replace the holder (see fig. 4)
      - ⇒ Replace the cover by setting it in position A (see fig. 3). Gently keep the cable taut while repositioning the cover.
      - ⇒ Replace the screws and tighten them.
      - ⇒ Insert the light tube and replace the light cover.
- Contact LUXO if any other problems arise.

### Specification of the product

T5 light tube, 22W, lifetime: 10,000 hours.  
Intensity of light at focus distance under the lens:  
280 mm under the lens: 3500 lux  
400 mm under the lens: 1500 lux

ART.: 05-900-695-00 Page 5 of 8  
BRU-303-0013 Rev.nr.: C

05-900-695-00

ART.: 05-900-695-00 Page 4 of 8  
BRU-303-0013 Rev.nr.: C

## Safety

Users of this form of lighting should be aware that the manufacturer of the product can only accept liability for the lighting when maintenance, repairs or changes are carried out by the manufacturer or by companies which the manufacturer has approved to carry out such work, and when components that influence the safety of the lighting are replaced with original spare parts. The lighting is not intended for use in areas where there is a risk of explosion.

## Fitting:

Circus lamps can be fitted to tables using the enclosed AH fitting.

## Precautions:

- Avoid spilling liquid on the lamp head
- NB! Never open the lamp head while the current is connected.

## Maintenance

NB! Start by switching off the power at the mains and removing the plug. In general, all repairs – including replacing fuses – must only be carried out by qualified electrical fitters or other similarly qualified professionals. The only maintenance routines not covered by this rule are changing the light tube and cleaning the lamp.

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