

Delasco Surgi-Cart[®]

parts lists and assembly instructions

Basic cart will be shipped separately by common carrier and can be expected to arrive within a week.

Accessories included here:

- 1- 734[®] or Hyfrecator[®] pole and mounting base
- 3- 3/4" plastic conduit straps
- 1- 3" wide aluminum strap, predrilled
- 1- 90 degree angle iron
- 1- metal mounting bracket, for sharps container
- 2- nuts and bolts for attaching mounting bracket to aluminum strap
- 1- sharps container, item # 6038
- 10- screws, phillips head


Tool required for assembly (not included): Phillips head screwdriver

Assembly:

Attach the three plastic conduit straps to the back of the cabinet, on the right side, (as seen facing the back), using the predrilled holes (labeled A) and 6 of the phillips head screws. Attach the 1" angle iron via the lowermost hole (B) and arrange the projecting portion so that it is parallel to the surface of the cart. The electrosurgical mounting pole should slip through the plastic straps, and rest on the projecting arm of the angle iron. If the pole does not slip through, the screws may be loosened and retightened after the pole is installed. Do not tighten too tightly, as the pole should be able to be twisted, to allow adjustment of the electrosurgical unit face angle during use.

The aluminum strap is attached to the left side via the three predrilled holes (C) and remaining phillips head screws, on the upper left of the cabinet, as viewed from the back with the attached washers between the back and the strap. If the metal mounting bracket is not already installed, install at the top with the two nuts and bolts.

The 6038 sharps container slips into the metal bracket. An electrosurgical unit can then be attached to the pole using that unit's wall bracket.

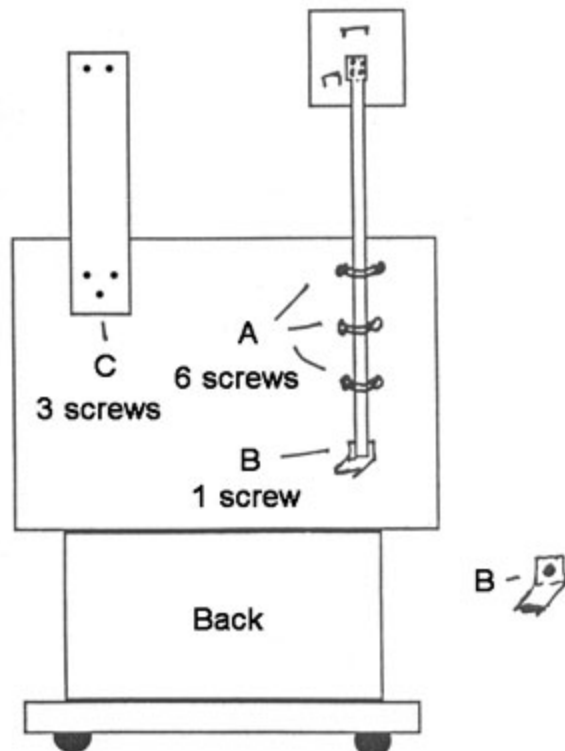

DELASCO
Dermatologic Lab and Supply Co,
Inc
608 13th Ave
Council Bluffs, IA 51501

1-800-831-6273